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On the right, a detailed view of a case is shown, including a '登録番号' (Registration Number) of 'bt32 x40', patient information, and a 'スライド一覧' (Slide List) with thumbnails of histology images labeled '1-1*', '1-2*', '2*', and '3*'. Each thumbnail has a 'リロード' (Reload) button and a green checkmark.

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The screenshot shows the Virtuoso web application interface. The browser address bar displays the URL: <http://180.43.171.113/virtuoso/Specimen.seam?conversationId=418>. The application header includes the name "Virtuoso" and the patient information "patient, btp32" and "btp32 x40".

On the left side, there is a sidebar with patient details:

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The main area displays a grid of 10 histology images, each with a "リリース済" (Released) status and a green checkmark. The images are labeled as follows:

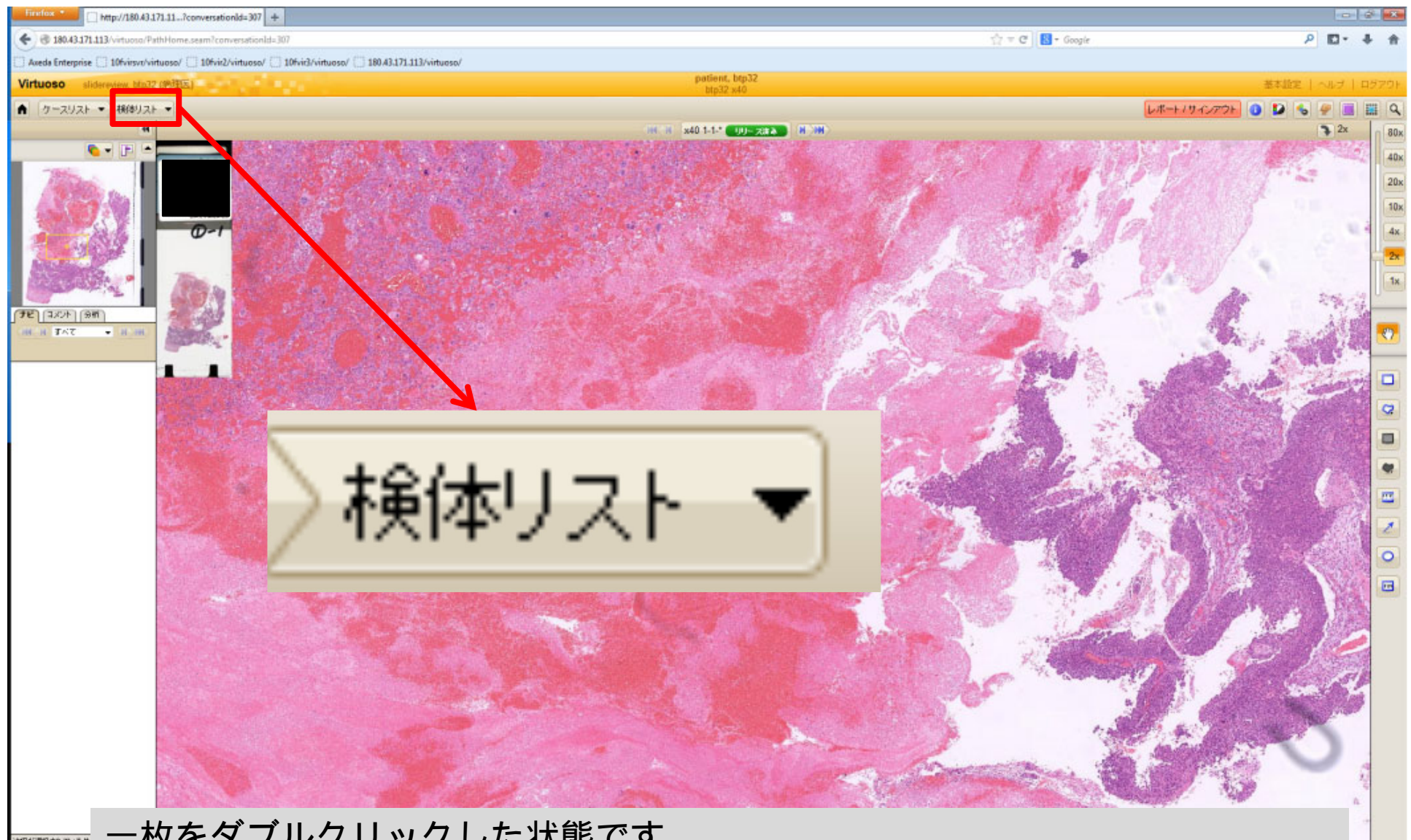
- 1-1.*
- 1-2.*
- 2.*
- 3.*
- 4-1.*
- 4-2.*
- 5.*
- 6-1.*
- 6-2.*
- 7.*
- 8.*

On the right side, there is a sidebar with the following sections:

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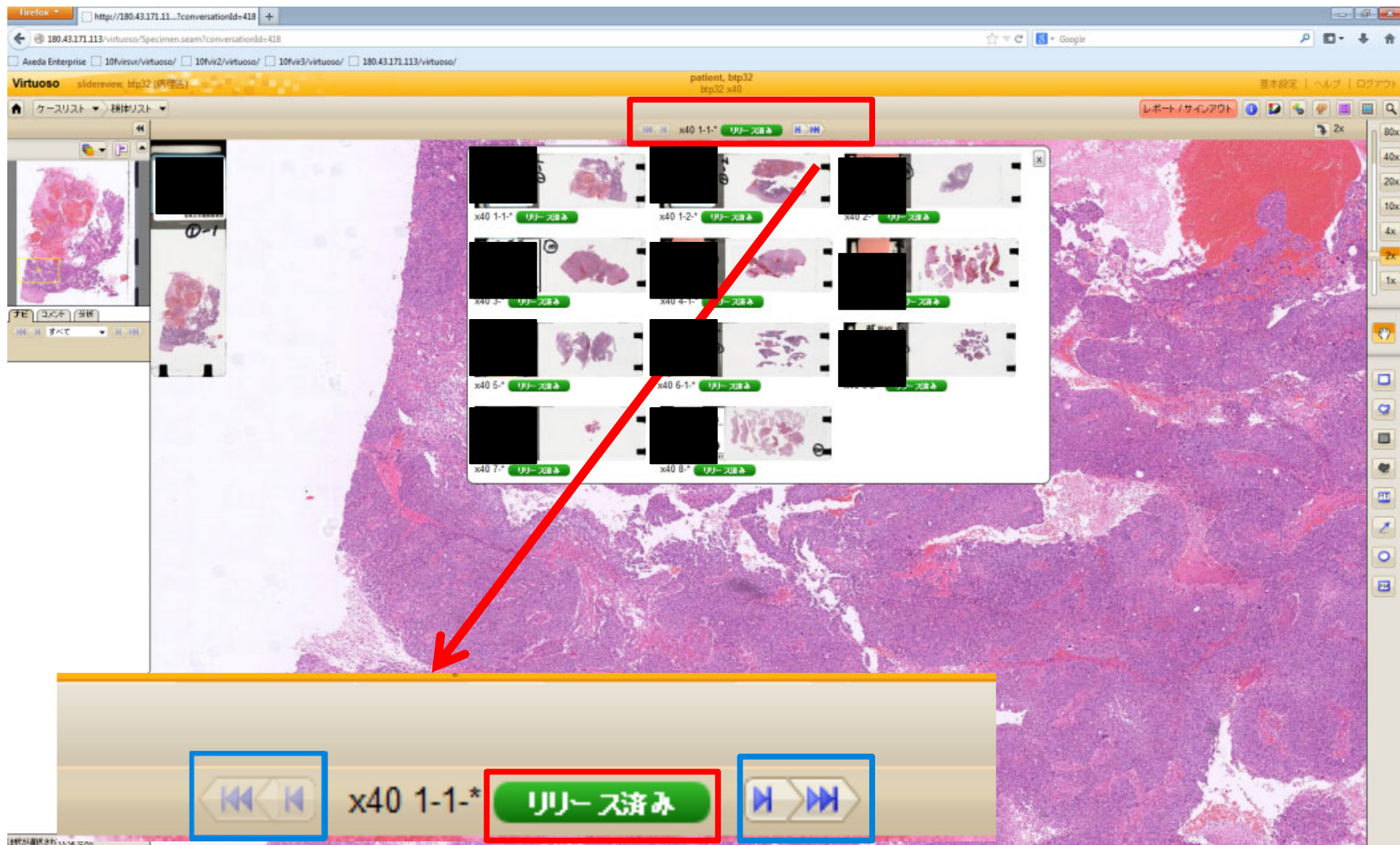
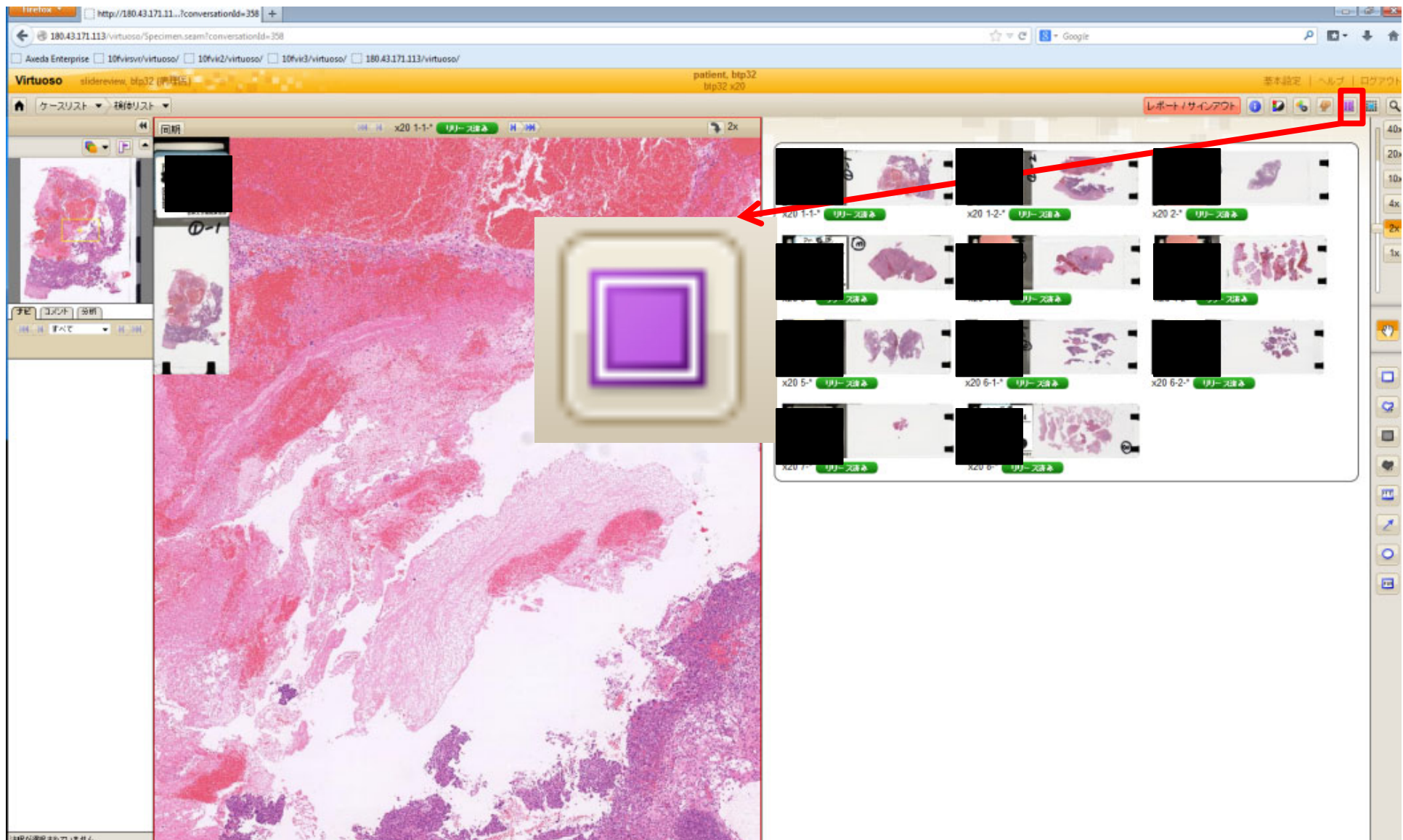
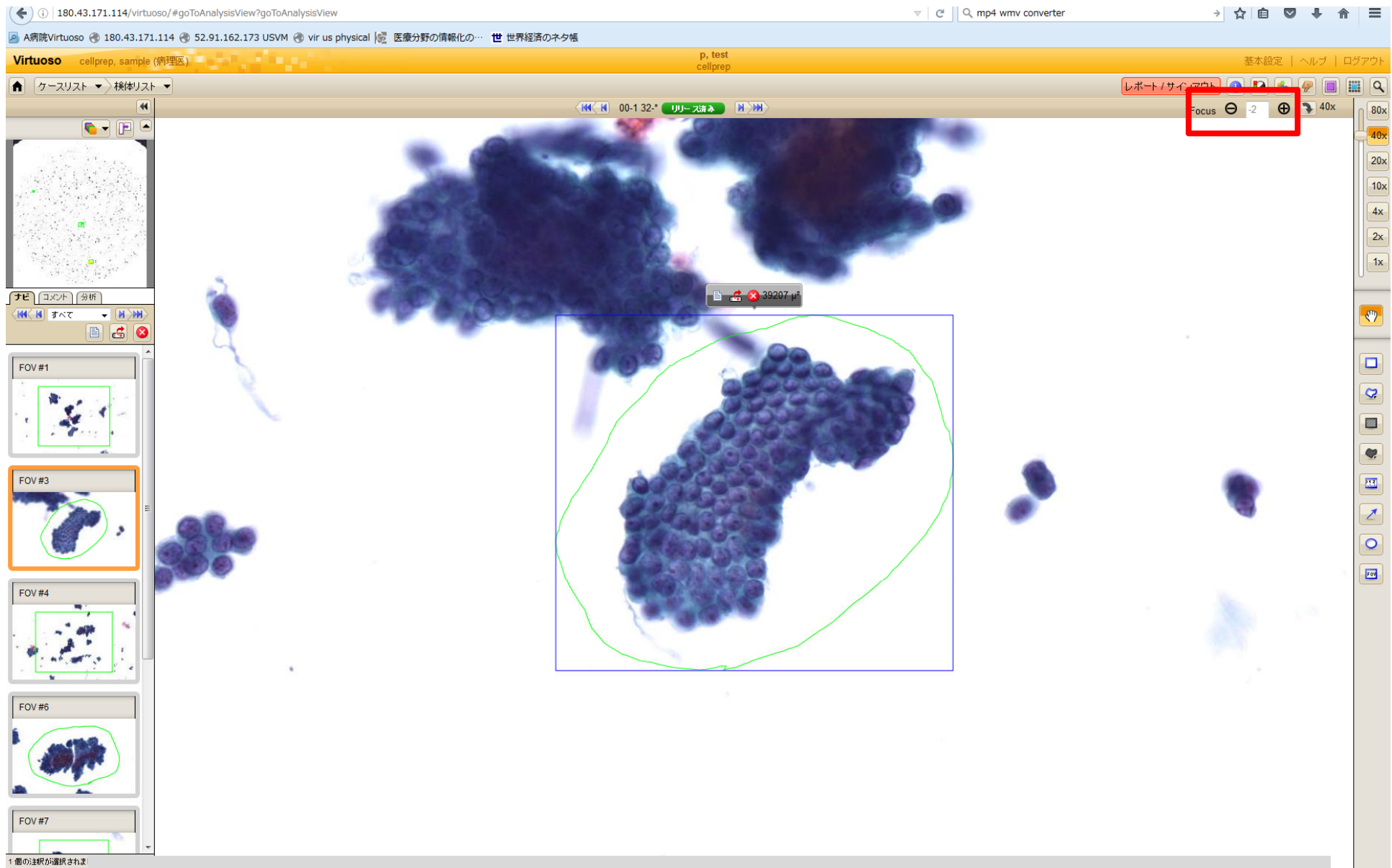
The screenshot displays the Virtuoso software interface. At the top, a browser window shows the URL `http://180.43.171.113/virtuoso/Path-Home.seam?conversationId=307`. The Virtuoso header is orange and contains the text "Virtuoso" and "slideview_btp32 (P.31/32)".

On the left sidebar, there are sections for "ケース" (Cases) and "その他" (Others). The "ケース" section includes "新しく作成" (New Case), "すべてのケース" (All Cases), "共有ケース" (Shared Cases), and "ケースの検索" (Search Cases). The "その他" section includes "ジョブキュー" (Job Queue), "サポート" (Support), and "Virtuosoについて" (About Virtuoso).

The main area features a table titled "ケースの共有 新しく作成" (Case Sharing New Case) with columns: "ケースID", "患者" (Patient), "患者ID", "登録日" (Registration Date), "タイプ" (Type), "サイト" (Site), "状態" (Status), "共有" (Sharing), and "レポート" (Report). The table contains two rows:

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Below the table, there are several panels for detailed case information:

- 登録番号: btp32 x40**: Includes fields for 患者名 (patient_btp32), 患者ID (6328521), 年齢性別 (Unknown), 本位組織タイプ (*), 登録日 (2014/04/04), and 割り当て先 (slideview_btp32 [btp32]).
- 検体の表示**: A section for displaying the specimen.
- 検体 (1)**: Shows "x40" with a "進行中" (In Progress) status.
- 検体の概要 x40:** A section for specimen overview.
- 生検順序 (1)**: Shows "btp32 x20".

On the right side, there is a vertical stack of image thumbnails for slide review. Each thumbnail shows a histological section with a green "リリース済み" (Released) label and a checkmark. The thumbnails are labeled "1-1*", "1-2*", "2*", and "3*" from top to bottom.

In the top right corner of the Virtuoso interface, there is a red rectangular box highlighting a small icon, which is the help/user guide icon.

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